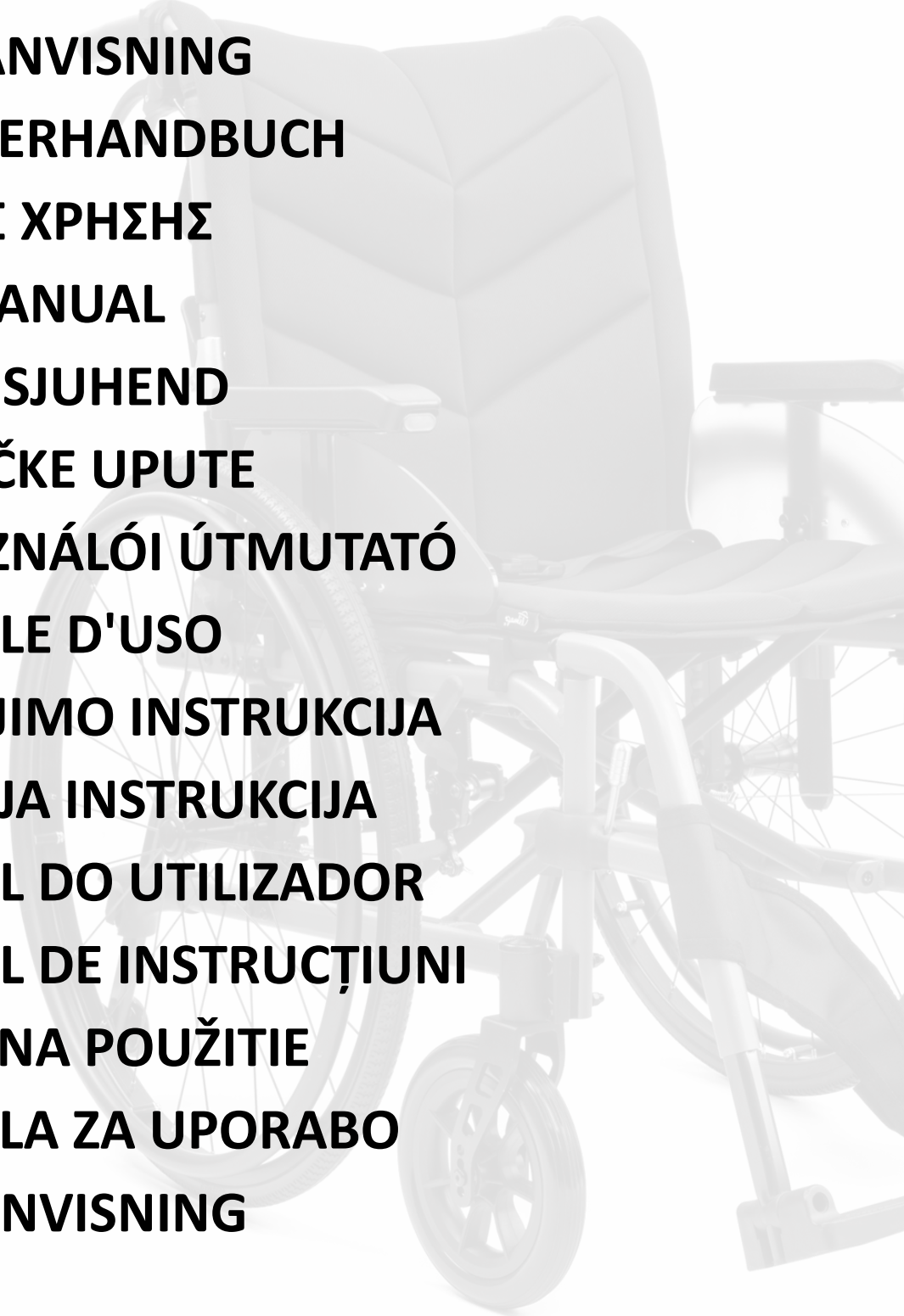
REF FRODO

- 
- PL INSTRUKCJA OBSŁUGI**
 - CS NÁVOD K OBSLUZE**
 - DA BRUGSANVISNING**
 - DE BENUTZERHANDBUCH**
 - EL ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ**
 - EN USER MANUAL**
 - ET KASUTUSJUHEND**
 - HR KORSNIČKE UPUTE**
 - HU FELHASZNÁLÓI ÚTMUTATÓ**
 - IT MANUALE D'USO**
 - LT NAUDOJIMO INSTRUKCIJA**
 - LV LIETOTĀJA INSTRUKCIJA**
 - PT MANUAL DO UTILIZADOR**
 - RO MANUAL DE INSTRUCTIUNI**
 - SK NÁVOD NA POUŽITIE**
 - SL NAVODILA ZA UPORABO**
 - SV BRUKSANVISNING**

PL INSTRUKCJA OBSŁUGI	4
CS NÁVOD K OBSLUZE	11
DA BRUGSANVISNING	17
DE BENUTZERHANDBUCH	24
EL ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ.....	31
EN USER MANUAL.....	38
ET KASUTUSJUHEND	45
HR KORSNIČKE UPUTE	51
HU FELHASZNÁLÓI ÚTMUTATÓ	57
IT MANUALE D'USO.....	63
LT NAUDOJIMO INSTRUKCIJA	70
LV LIETOTĀJA INSTRUKCIJA.....	76
PT MANUAL DO UTILIZADOR	82
RO MANUAL DE INSTRUCȚIUNI.....	89
SK NÁVOD NA POUŽITIE	96
SL NAVODILA ZA UPORABO	103
SV BRUKSANVISNING	110

ELEMENTY WÓZKA

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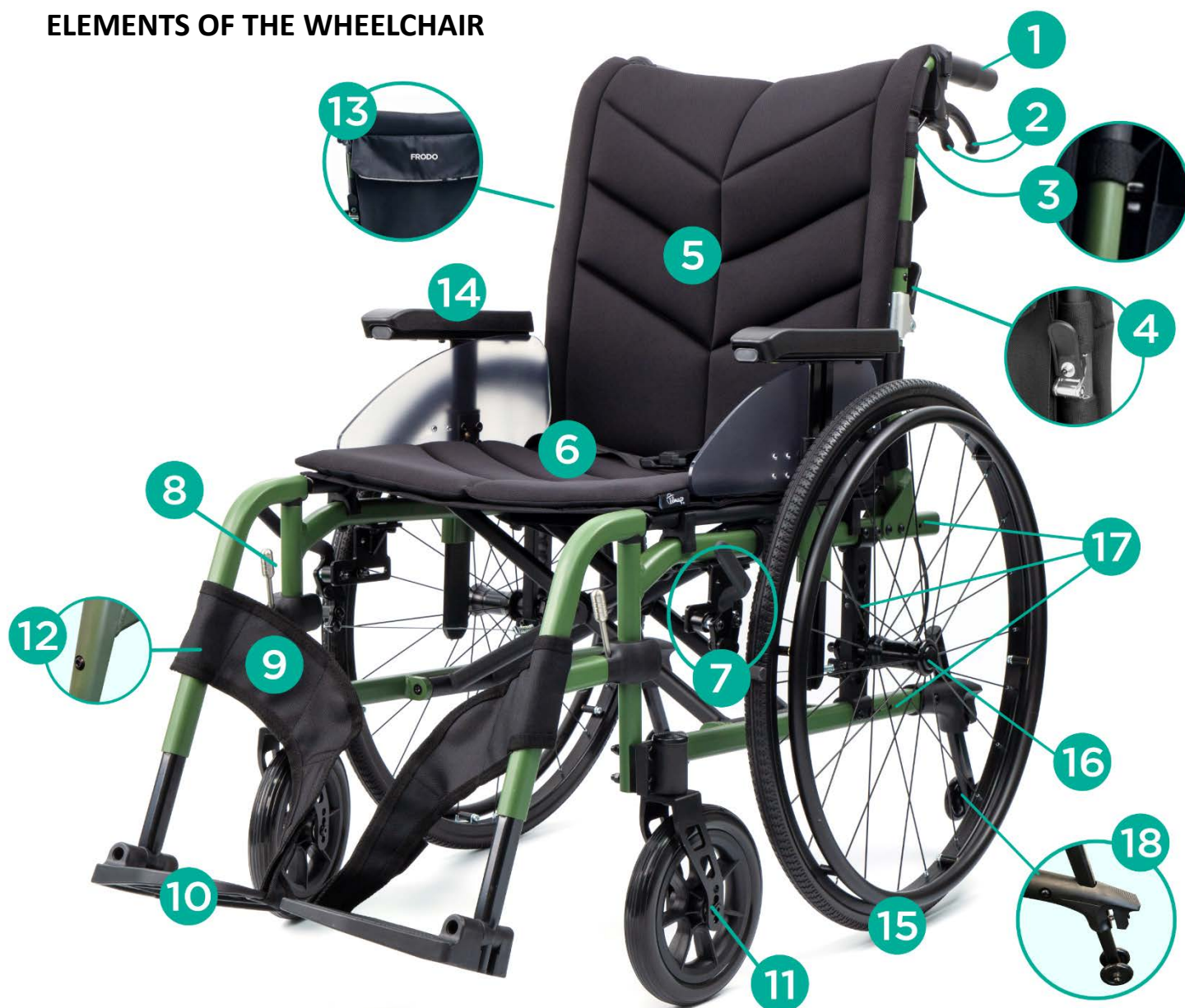
ELEMENTS OF THE WHEELCHAIR

- | | |
|-----------------------------|--|
| 1. Handgrip | 11. 8" swivel, solid front wheel with centre of gravity adjustment |
| 2. Attendant brake | 12. Footrest length adjustment |
| 3. Handle adjustment knob | 13. Backrest pocket for goods |
| 4. Backrest folding clamp | 14. Adjustable armrest |
| 5. Adjustable backrest | 15. 24" solid, rear wheel with handrim |
| 6. Seat* with security belt | 16. Quick-release function button |
| 7. Parking brake | 17. Centre of gravity adjustment (all directions) |
| 8. Footrest lever | 18. Anti-tip wheel with length adjustment |
| 9. Calves protective strap | |
| 10. Footplate | |

* available in size 44, 46 and 49 cm

1. Characteristics

The wheelchair is made of a lightweight aluminum frame that features multiple adjustment options to match user's specific needs:

- a) complex centre of gravity adjustment on rear wheels, allowing adjustment in any direction (up/down/front/back).
- b) standard centre of gravity adjustment/wheel angle adjustment on front wheels and quick release function on rear wheels.
- c) Height adjustment for handles/armrests/backrest and length adjustment for anti-tip wheels/footrests.

It is equipped with parking brakes, attendant brakes, security belt, backrest folding function and wheels: rear 24" (solid), front 8" (solid, swivel). Additionally, wheelchair comes in set with breathable mesh upholstery (with pocket on backrest), removable armrests with height adjustment, anti-tip wheels and footrest with calves protective straps.

The set includes: wheelchair frame with wheels, upholstery with pocket for goods, footrests with protective straps, anti-tip wheels, armrests, set of keys for assembly/dismantling.

2. Application

The wheelchair is designed for people who have problems with unassisted moving and for patients with dysfunctions and reduced mobility. This self-propelled wheelchair can also be pushed by an accompanying person. Thanks to the ability to adjust the settings to the individual needs of the user, the wheelchair provides better control, greater stability, and a more dynamic, active ride on everyday bases. The wheelchair is suitable for indoor and outdoor use.

3. Assembly

Step 1. Unfolding the wheelchair

Place the wheelchair on the ground. Unfold the wheelchair by the armrests. Then press the sides of the seat so that all the elements fit together.

Step 2. Backrest and seat

Lift the backrest upwards and lock it with backrest clamps [4] which are located below the handles. Fix the upholstery with pocket on the wheelchair seat and backrest.

Step 3. Footrests

Fix the footrest on the outer side of the frame. Then turn the footrest inwards (press the footrest lever at the same time) until it is locked.

Step 4. Armrests

The armrests can be removed or height adjusted. To raise/lower the armrest, press the button located on the frame, under the armrest.

Step 5. Height adjustment of the footrests and handles.

- Adjust the footrests using set of assembly keys. To adjust, unscrew the screw [12] located on the footrest frame. Enough space should be left between the

footplates and the ground as footplates set too low may cause difficulties in climbing hills and curbs. Adjust the footplates so that the angle between trunk and thighs is 90° (optimum sitting position for the user).

- Handle height can be adjusted according to accompanying person needs. To adjust height, unscrew mounting knob [3] located under attendant brakes [2]. Select desired height and tighten the knob. Make sure both handles are on the same height.

FOLDING AND BACKREST FOLDING FUNCTION

- Swing away the footplates and set them in vertical position or remove. Lift the seat upwards by grabbing material handle on the seat. Squish wheelchair from both sides inwards to endure proper folding. Detach the seat and backrest upholstery with bag if needed.
- To fold the backrest, simultaneously press locking elements which are located under the hand grips at the back of the backrest and pull the backrest downwards.

BACKREST & HANDLE HEIGHT ADJUSTMENT

Note! Backrest can be only adjusted after raising/lowering handles height.

Backrest can be adjusted according to the user's needs. To adjust backrest, remove first upholstery. Then unfasten top velcro on the back of the wheelchair and move backrest higher. Fasten Velcro afterwards and mount upholstery again.

PARKING BRAKE

To lock the wheels, push the lever forward until the wheel is fully blocked (separately for left and right wheel). Please, note that the parking brake only serves to lock the wheels when the wheelchair is at a complete stop.

ATTENDANT BRAKES

The levers of the brakes are located behind the backrest, under both grips.

- In order to stop the wheelchair, squeeze the lever of the brake handle. To release the brake, let go of the levers. Perform this activity smoothly and with appropriate force.
- The brakes are equipped with parking locks. In order to use them, squeeze the top lever of the brake handle till you hear "click" sound. To release parking brakes, squeeze the bottom lever of the brake handle.

PARKING BRAKE ADJUSTMENT

The correct lock of the parking brake depends on its adjustment and position. The handle of the wheel lock should be positioned so that the brake shoe blocks the wheel firmly and the lever does not face excessive resistance.

- In case of excessive resistance, pull the handle slightly forward by twisting off the locking screws which are located on the parking brake. Set the brake shoe in such position that it blocks the wheel easily without excessive resistance.
- When the parking brake shoe does not touch or lock the wheel sufficiently, follow the above instruction by moving the handle in the opposite direction.

CENTER OF GRAVITY ADJUSTMENT ON FRONT/REAR WHEELS

(ADJUSTMENT FOR PATIENT AFTER AMPUTATION IN THE LOWER LIMBS AREA)

Note! For professional adjustment, ask your retailer or other person who has appropriate knowledge and qualifications. Do not adjust inclination angle of the seat by yourself.

- To adjust the centre of gravity on front wheels, unscrew bolts in right and left wheel with assemble keys added to wheelchair set (picture 3). Then, set the seat inclination by moving fixing axis of the wheel downwards or upwards (the same on both wheels).
- To adjust the centre of gravity on rear wheels, unscrew bolts in right and left wheel with assemble keys added to wheelchair set (picture 1 and 2). Then, set the seat inclination by moving fixing axis of the wheel front, back, downwards or upwards (depending on patient's needs, the same in both wheels).

ANGLE ADJUSTMENT ON FRONT WHEELS

Note! For professional adjustment, ask your retailer or other person who has appropriate knowledge and qualifications. Do not adjust front wheel angle of the wheelchair by yourself.

- To adjust front wheels angle, unscrew the bolts located on the inner side of the wheelchair (picture 4) using assemble keys. Take out cap and washer. Grab the front wheel slightly and push bolt from inner side towards front. Grab the bolt cap on the outer side of the wheelchair and turn left/right, setting new angle according to user's needs (picture 5). Put the washer and cap again, tightening it with a assemble key.

4. Before use

Step 1. Checking the condition of the wheelchair

Check the condition of the brakes, construction and tires. Check if all the components are assembled and adjusted properly.

Note! Do not use the wheelchair if any defects are noticed.

Step 2. Using parking brakes

Lock the wheels when getting in or out of the wheelchair.

Step 3. Footplates

The footplates should be set in vertical position when getting in or out of the wheelchair. Do not use the footplates as a support while getting in or out of the wheelchair, due to the risk of falling. After transferring to the wheelchair, unfold the footplates.

Note! Do not stand on the footplates due to the risk of damage.

Step 4. Getting in the wheelchair

The wheelchair should be located as close to the patient as possible. The wheelchair should touch the bed or other seat being used. If the user cannot change their seat independently, transfer the user to the wheelchair.

Step 5. Position of the user

The user should sit in an upright position and their knees should be bent at the angle of 90°.

The wheelchair is ready to use.

5. Technical data

Seat width: 44/46/49 cm	Seat depth: 43-48 cm
Overall depth: 105 cm	Backrest height from the seat: 43-53 cm
Overall width: 62/64/67 cm	Armrest height: 18,6-27 cm
Seat height from the ground: front: 48-50 cm rear: 41,5-49 cm	Folded dimensions: 44/46 cm – 75 x 29 x 73,5 cm 49 cm – 75 x 29 x 76 cm

Overall height: 93,5-103,5 cm	Weight: 11,30-11,50 kg
Maximum capacity: 125 kg	

6. Notes

Timago International Group does not bear responsibility of improper usage of the product, failure to comply safety regulations and misuse.

7. Cleaning and maintenance

Carry out the maintenance of the wheelchair components regularly:

- The upholstery and the cushion can be cleaned with damp (not wet) cloth and cold or warm water, without the use of detergents. Difficult stains can be cleaned with a sponge. Do not soak the upholstery too much. Dry away from heat sources.
- Plastic elements should be cleaned with damp cloth possibly with the use of appropriate detergents for plastic elements. In such case follow the safety measures and instructions given by the manufacturer.

8. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

9. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

10. Medical device incidents

Any serious incident involving a medical device must be reported to the manufacturer and to the competent authority of the European Union member state where the user or patient resides.

11. Labels

REF	Reference number		Manufacturer
LOT	LOT number	UDI	UDI code
SN	Serial number	MD	Medical device

	Note		Please read the instruction
	The product meets the legal requirements of the European Union for medical devices (MDR 2017/745)		

Our complete line of products is available from our distributors. Learn more at:
www.timago.com. Thank you for choosing Timago!

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