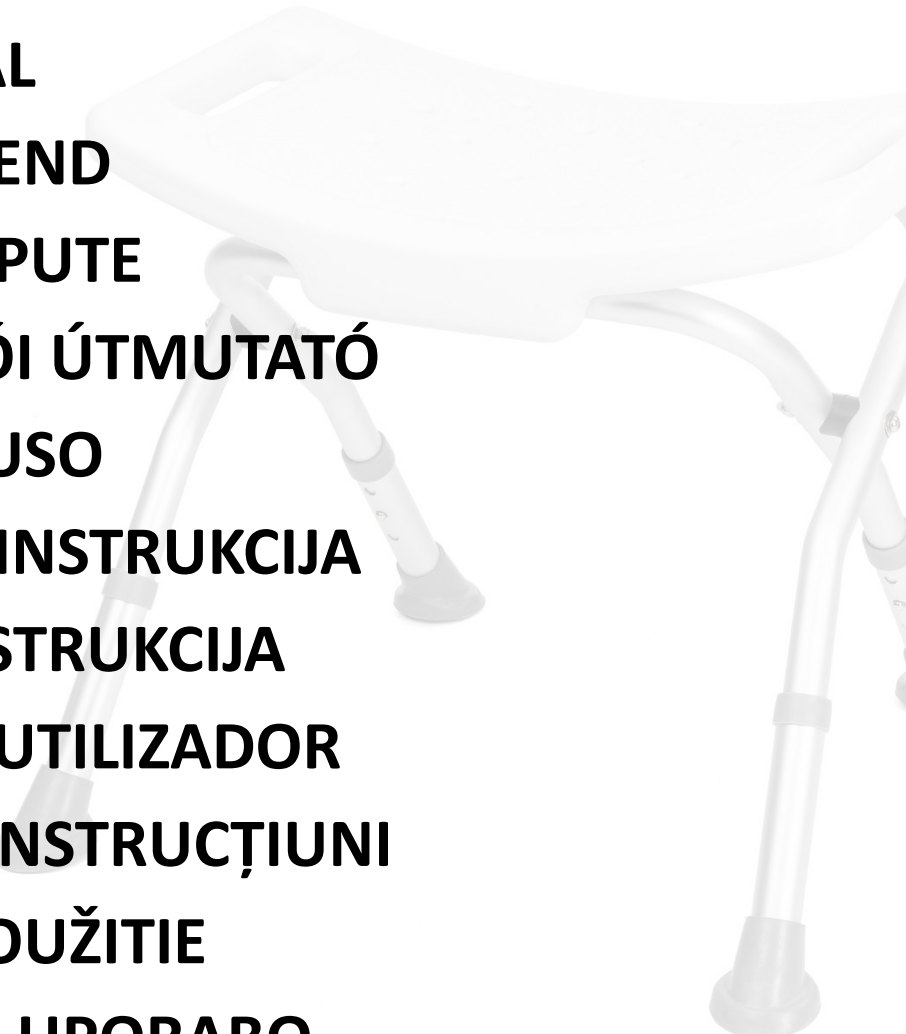
REF TGR-R KP 540

- PL INSTRUKCJA OBSŁUGI**
CS NÁVOD K OBSLUZE
DA BRUGSANVISNING
DE BENUTZERHANDBUCH
EL ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
EN USER MANUAL
ET KASUTUSJUHEND
HR KORSNIČKE UPUTE
HU FELHASZNÁLÓI ÚTMUTATÓ
IT MANUALE D'USO
LT NAUDOJIMO INSTRUKCIJA
LV LIETOTĀJA INSTRUKCIJA
PT MANUAL DO UTILIZADOR
RO MANUAL DE INSTRUCTIUNI
SK NÁVOD NA POUŽITIE
SL NAVODILA ZA UPORABO
SV BRUKSANVISNING
- 

PL INSTRUKCJA OBSŁUGI.....	4
CS NÁVOD K OBSLUZE.....	7
DA BRUGSANVISNING.....	10
DE BENUTZERHANDBUCH.....	13
EL ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	16
EN USER MANUAL	20
ET KASUTUSJUHEND	23
HR KORISNIČKE UPUTE	26
HU FELHASZNÁLÓI ÚTMUTATÓ	29
IT MANUALE D'USO	32
LT NAUDOJIMO INSTRUKCIJA	35
LV LIETOTĀJA INSTRUKCIJA	38
PT MANUAL DO UTILIZADOR.....	41
RO MANUAL DE INSTRUCȚIUNI	44
SK NÁVOD NA POUŽITIE.....	47
SL NAVODILA ZA UPORABO.....	50
SV BRUKSANVISNING.....	53

ELEMENTY TABORETU

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ELEMENTS OF THE CHAIR

1. Seat
2. Side handle
3. Mounting clamp
4. Height adjustment
5. Leg with anti-slip tip

1. Characteristics

Shower chair is made of aluminium frame and wide seat made with durable PVC. The chair has a height adjustable legs, side handles and anti-slip rubber tips that prevents the chair from sliding on wet surfaces. Specially designed chair base allows for easy folding and quick storage. The chair is additionally supplied with Tool-Free (TF) assemble system.

The set includes: seat with foldable chair base, height adjustable legs (4 pcs).

2. Application

The shower chair is equipment designed to help the user to perform hygienic activities. The chair is recommended for people with reduced mobility which prevents or hinders the unassisted performance of daily living activities. The chair can be used in the shower, in the bath, in bathrooms and other humid rooms at home and in hospitals.

3. Assembly

Step 1. Take out the chair and its legs from box.

Step 2. Mount the 4 legs on the base. Secure them with locking pins on desired height. Make sure all legs are on the same height!

Step 3. Grab the base frame and mount the remaining tube into silver slot on the bottom of the seat (picture 1).

HEIGHT ADJUSTMENT

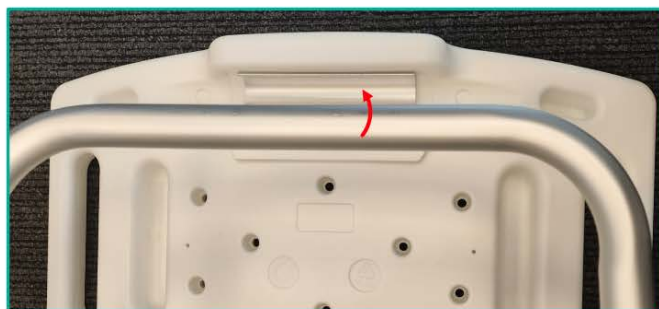
To adjust the height of the shower chair, press locking pin located on the legs, select appropriate height and lock in the position. The legs are adjustable in 2,5 cm increments.

Note! The pins must be set on the same level in all legs.

FOLDING

To fold the chair, put it on the floor, facing seat downwards and put the foot on the seat, grabbing the opposite base frame to the one mounted in silver slot (picture 2) and pull up.

Note! Do not apply excessive force when pressing down on the seat.

Picture 5 | Picture 2

4. Technical data

Overall width: 54-56 cm	Seat dimensions: 50 x 25 cm
Overall length: 44-50 cm	Seat height from the ground: 36,5-41 cm
Height adjustment: 40-44,5 cm	Weight: 2,80 kg
Maximum capacity: 136 kg	Folded dimensions: 50 x 11 x 67 cm <i>dimensions measured without the legs</i>

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Maintenance

- Clean the shower chair regularly using only water, sponge and mild detergents for plastic materials such as soap. Dry thoroughly.
- **Do not use** for cleaning: strong detergents, abrasive agents, bleach, petrol or solvents.
- **At least once every three months check:** condition of locking pins, condition of tubes (if cracks). It is recommended to check the rubber tips regularly. In case of worn-out, replace the tips with the new ones.

7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Medical device incidents

Any serious incident involving a medical device must be reported to the manufacturer and to the competent authority of the European Union member state where the user or patient resides.

10. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The product meets the legal requirements of the European Union for medical devices (MDR 2017/745)		

Our complete line of products is available from our distributors. Learn more at: **www.timago.com**. Thank you for choosing Timago!

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