USER MANUAL

Foam anti-decubitus mattress with washable vapor-permeable cover

REF: TriMATE Care, type: GOFR, size: 200x90x15 cm

Before using the product, read these instructions. The instruction contains information necessary for the safe and proper use of the product, how to maintain and clean it. Use of the product not in accordance with the rules of the instructions for use can risk serious complications. The instruction should be kept in an easily accessible place.

1. Characteristic

The mattress is made of polyurethane foam, and its waffle-shaped design provides anti-decubitus properties. The mattress is equipped with a zippered washable vapor-permeable cover. It includes 3 different layers of foam (28D bottom; 25D middle, 30D top) that provides optimal stabilization, ventilation and pressure redistribution.

2. Application

The anti-decubitus foam mattress, thanks to its design, allows constant access of fresh air, which, combined with the proper distribution of pressure and adaptation of the mattress to the body of use, prevents the formation of pressure sores, and also helps in the treatment of ones already formed.

3. Contraindications for use:

- Do not use the product mismatched with the bed frame.
- Do not use the product other than its intended use.
- Do not use the product near an open flame or on a hot surface. It may seriously injure the user and lead to serious health damage.

4. Notes

Manufacturer does not bear responsibility of improper maintenance of the product, failure to comply safety regulations and misuse.

- The skin at the site of connection should be clean.
- It is necessary to regularly check the condition of the patient's skin.

- The position of users should be changed regularly to avoid the formation of sores/wounds.
- Do not use sharp objects and equipment that can damage the cover.
- The mattress should be matched to the size of the bed.
- Maintain high hygiene of the mattress cover: clean and/or wash.

If the product is no longer suitable for use, it should be disposed of. Before disposal, the product should be cleaned to eliminate the risk of contamination.

If there are any adverse symptoms resulting from the use of the device, report it to the manufacturer and to the competent authority of the Member State where the user or patient resides.

5. Unpacking and fitting

- When you receive the shipment with the mattress, open it carefully so as not to damage the mattress.
- Lay the mattress on a flat surface and wait for all the deformations caused during transportation to return to its original shape.
- The mattress should be fitted to the patient's bed to fully fulfill its functions.

6. Technical data

PARAMETERS	TriMATE Care
Length	200 cm
Width	90 cm
Height	15 cm
Capacity	250 kg

7. Cleaning and maintenance

- Do not wash the foam mattress.
- Wash the cover after removing the foam insert in a mild detergent at temperatures up to 90°C.
- The cover can be sterilized at temperatures up to 120°C.
- The cover can be disinfected with up to 60% alcohol solution.
- Do not use bleach or chemicals.
- Do not clean chemically.
- Do not use chemicals containing chlorine.
- Drying in an unfolded state is recommended.

- Do not allow heavy soiling.

8. Storage and transport conditions.

Product should not be exposed to direct sunlight, water or moisture.

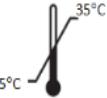
9. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

COMPLAINTS ARE NOT SUBJECT TO:

- Mechanical damage (abrasions, scuffs).
- Damage resulting from natural use.
- Defects resulting from improper sizing of the mattress to the bed.
- Defects resulting from failure to observe the conditions of use or the method of cleaning and maintenance.
- Discoloration of the cover.
- Comfort of use.

10. Labels

	Reference number		Manufacturer
	LOT number		Unique device identifications
	Serial number		Medical device
	Note		Please read the instruction
	Use by date		Permissible storage temperature
	Store in a dry place		Protect from sunlight

	Packaging is recyclable		The packaging should be disposed of in the trash
	The product meets the legal requirements of the European Union for medical devices (MDR 2017/745).		

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