USER MANUAL

Bedside rail

Product code: HandyGrip


1. Characteristics

Bedside rail is made of durable, powder-coated steel. It has a foam handle and a pocket for small items. The rail comes with screws and a safety belt, providing the user with additional stability and comfort.

The set includes: bedside rail with pocket, screws, security belt.

Elements of the rail:

- A. Handle
- B. Frame folding mechanism
- C. Pocket for goods
- D. Lower frame
- E. Upper frame

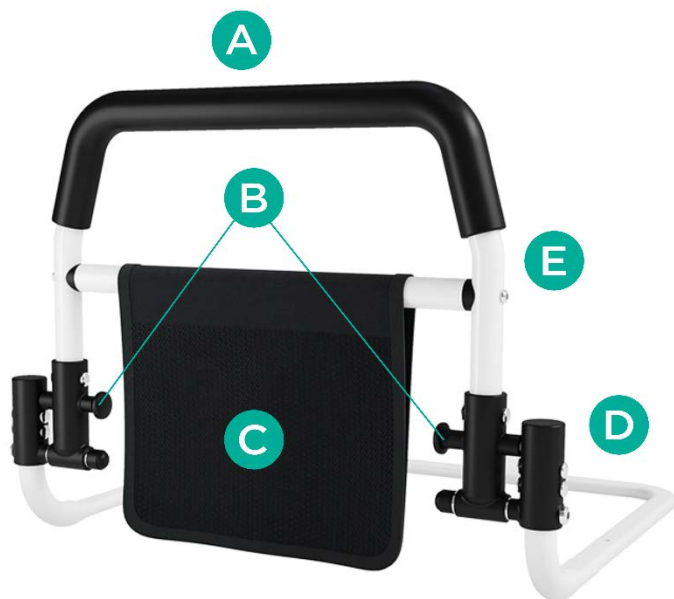2. Application

The bedside rail is designed to provide a support for standing up or switching from laying to sitting position in bed. The rail is recommended for people with reduced mobility which hinders the unassisted performance of daily living activities. The bedrail rail is recommended for use both during care in medical facilities and at home.

3. Assembly

Step 1. Take out bed rail out of box and unfold it till you hear click sound.

Step 2. Place the bedside rail lower frame on the bed base, with upper frame facing outwards.

For extra security, bedside rail should be mounted by using of two methods:

- Screws attached to the set: place lower frame on the bed base and secure it using screws using screw or driller.
- Security belt added to the set: place lower frame on the bed base and secure it with belt around the bed base.

Step. 3 Place mattress on the bed, covering lower frame of the rail.

Bedside rail is ready to use.

FOLDING/UNFOLDING

Rail can be folded, allowing user for easier storage or transportation if needed:

- To fold the rail for the transportation grab frame folding mechanism (B) and pull outwards, lowering upper frame of the rail.
- To unfold the frame, grab the upper frame by handgrip and put it up till

4. Technical data

Overall height	44 cm
Overall width	57 cm
Overall depth	50 cm
Handle length	38 cm
Dimensions after folding	50 x 57 x 14,5 cm
Lower frame tube thickness	1,2 cm
Weight	2,00 kg
Max. weight capacity	136 kg

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Maintenance

Clean the frame and bag regularly using damp cloth, without use of ingredients. Dry thoroughly. Difficult stains on the bag can be cleaned with sponge.

7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Medical device incidents

Any serious incident involving a medical device must be reported to the manufacturer and to the competent authority of the European Union member state where the user or patient resides.

10. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The product meets the legal requirements of the European Union for medical devices (MDR 2017/745).		

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!

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